— MOBILE CARE BED —

Assembly Instructions
Mobiles Pflegebett
Montageanleitung

IMPORTANT INFORMATION CONCERNING THIS PRODUCT TO PREVENT SERIOUS OR FATAL INJURIES

Keep these instructions for future reference

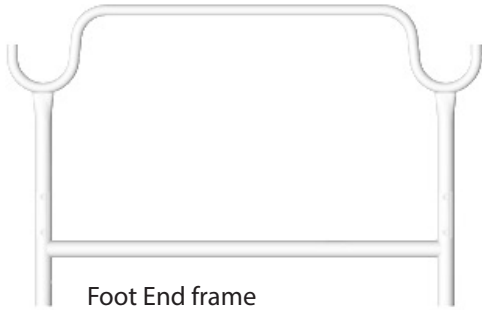
1. Read carefully and fully understand the manufacturer's assembly.
2. Care and special attention should be used where a warning is indicated.
3. DO NOT use substitute parts. Contact the manufacturer or dealer for replacement parts.
4. Metal parts could have sharp, pointed or jagged edges. Remove these with a file or an emery board.
5. The adjustable back and head rest should not be operated by the patient and should only be adjusted by the care team.
6. The care team should take care when adjusting the head and back rest as it may present a pinch hazard.
7. Maximum weight tolerance is 500 lbs.
8. All 4 wheels should be locked into place when the Mobile Care Bed is not intentionally being moved.
9. The Mobile Care Bed is designed for indoor use or for use inside of a tent. When not in use should be stored in climate controlled dry storage.
10. **STRANGULATION HAZARD** – Never attach or hang items to any part of the bed that are not designed for use with the bed; for example, but not limited to, hooks and belts.

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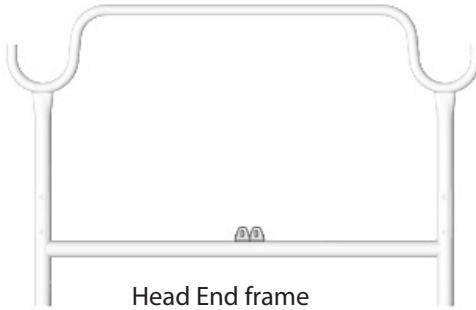
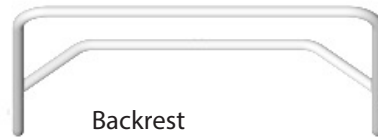
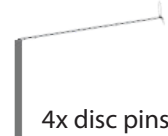
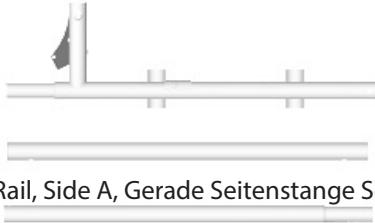
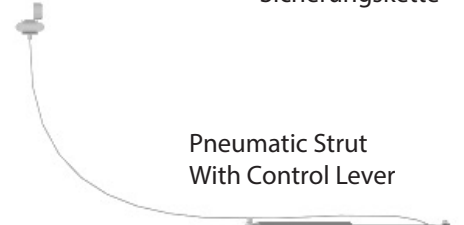
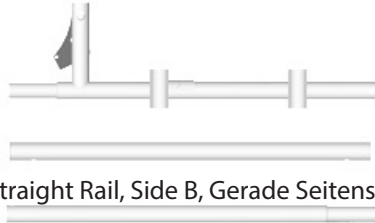
Pneumatic Strut
With Control Lever

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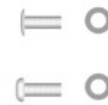
STEP 1. PREPARE PARTS FOR ASSEMBLY

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- 1.1. Remove the discs from each of the bed end frames and set to the side for later use.
- 1.2 Lay out the head and foot end frames about 6 ft. apart from one another.

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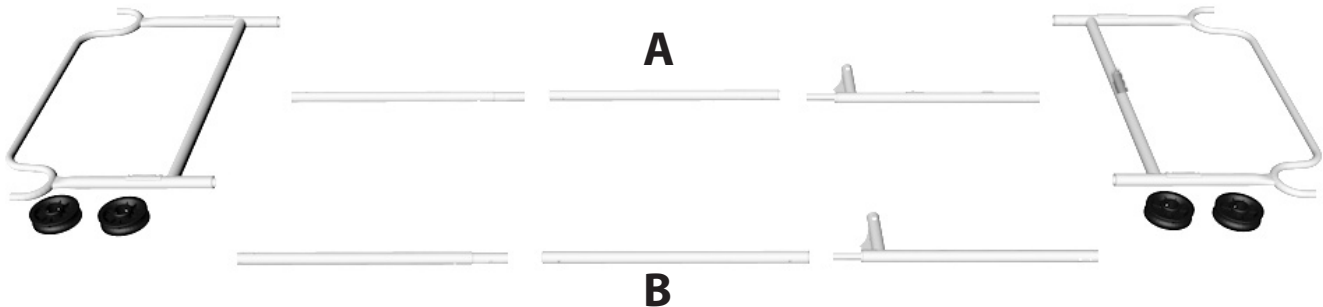
- 1.3 Layout the head swaged rails (a and B), straight rails (a and B)on floor, and foot swaged rails (a and B).

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
2.1 Attach head swaged side rail A to straight rail A.

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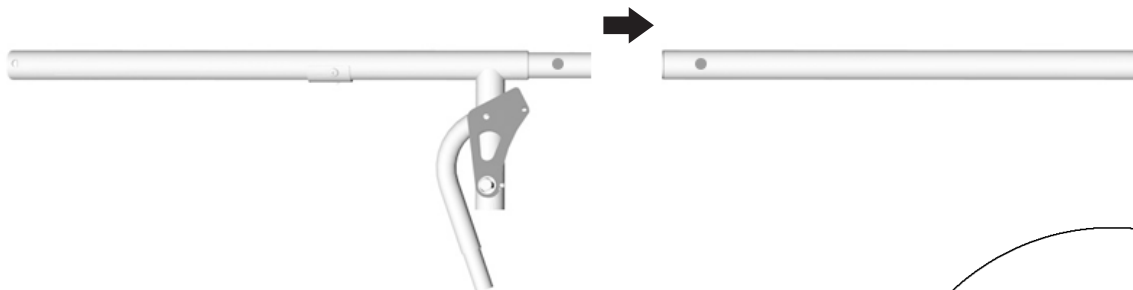2.2 Attach foot swaged side rail A to straight rail A.

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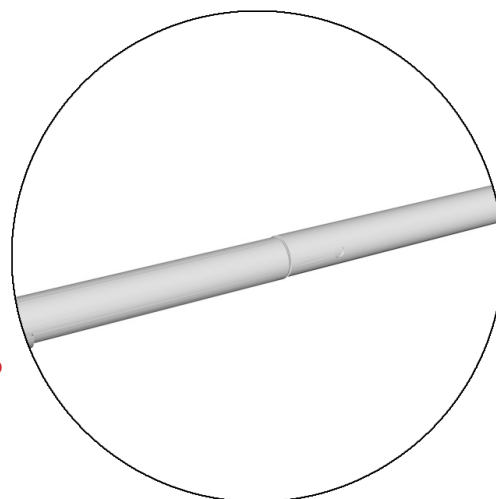
Make sure the rails slide into one another completely and that the pin has popped into place.

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STEP 3. SECURING DISC TO ASSEMBLED SIDE RAILS

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3.1 Holding the head swaged rail A, press pin A down and hold.

3.2 Line up pin with hole in disc, insert into disc. Let pin go.

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
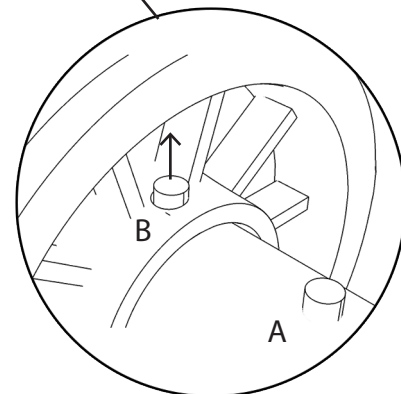
Pin (B) is only securely locked in when you hear a 'click' sound. Make sure pin (B) has popped up before moving on.

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
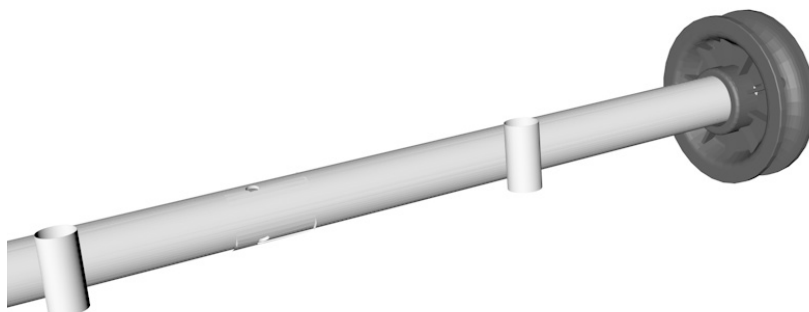
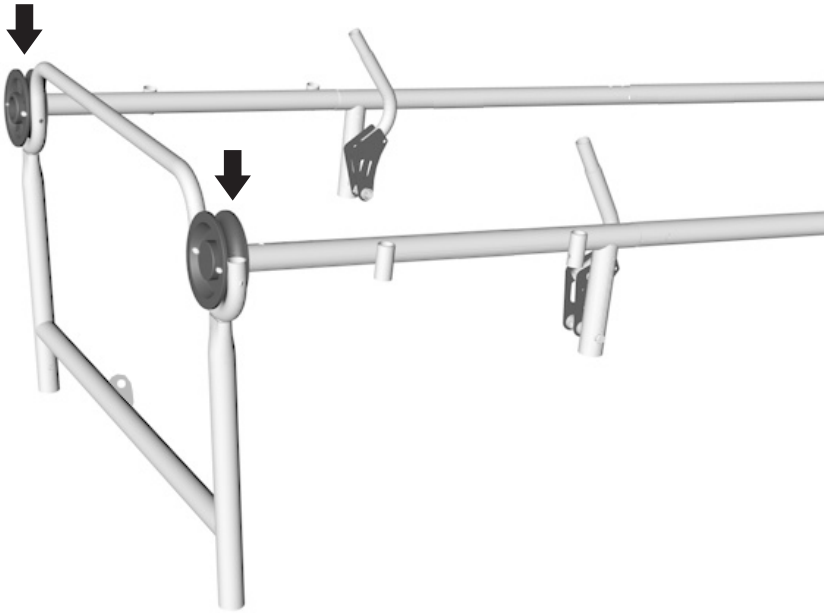
Make sure the pin holes in the disc line up with the holes in the end frames.

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

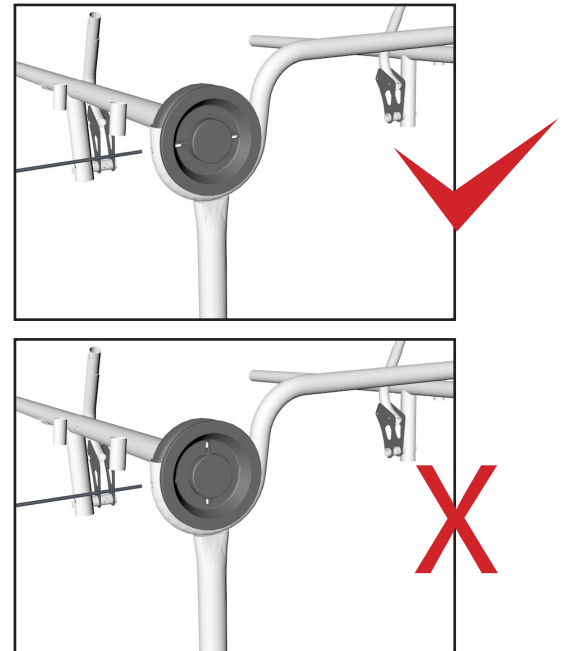
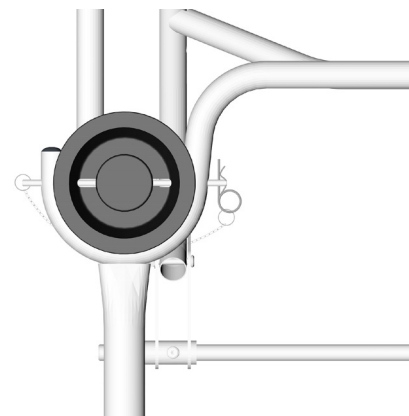
5.1 Line up the back rest support bar, and slide through each hole on the head swaged rail support rails.
5.2 Using bolt and wing nut, hand tighten

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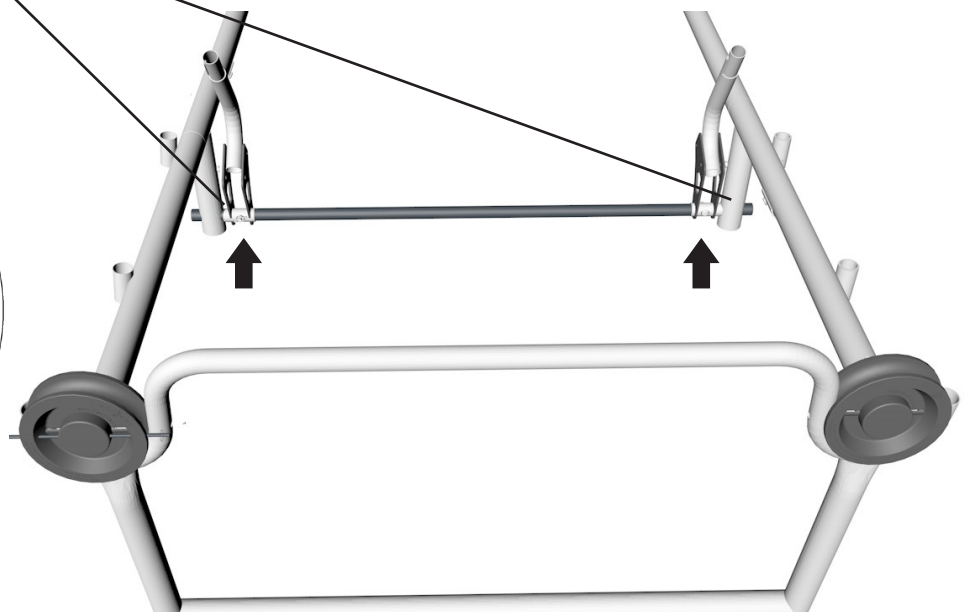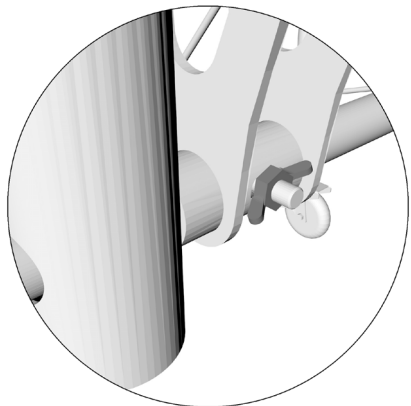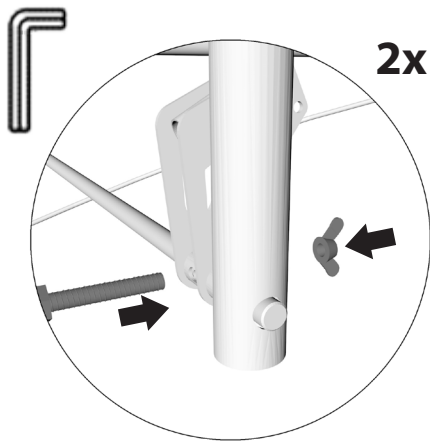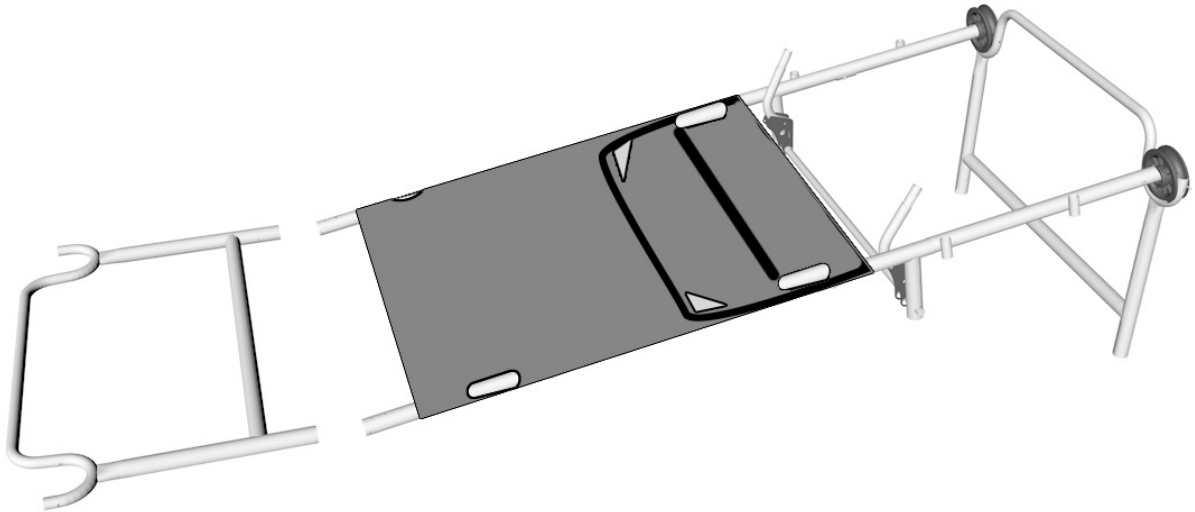
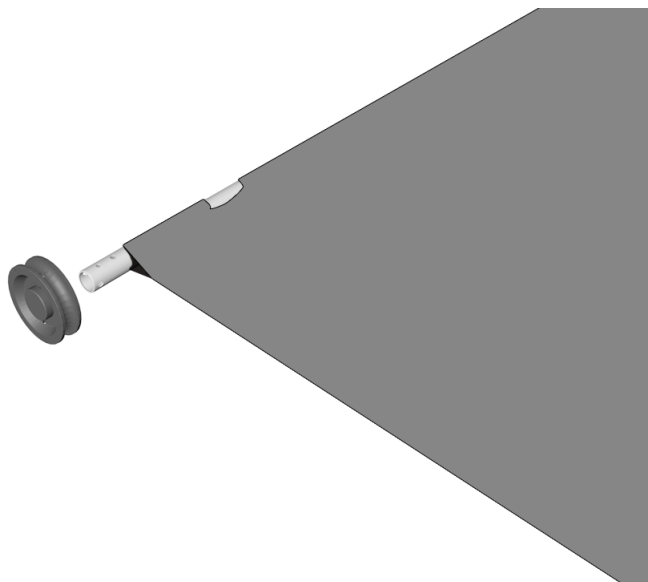
Pin is only securely locked in when you hear a 'click' sound. Make sure pin has popped up before moving on.

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
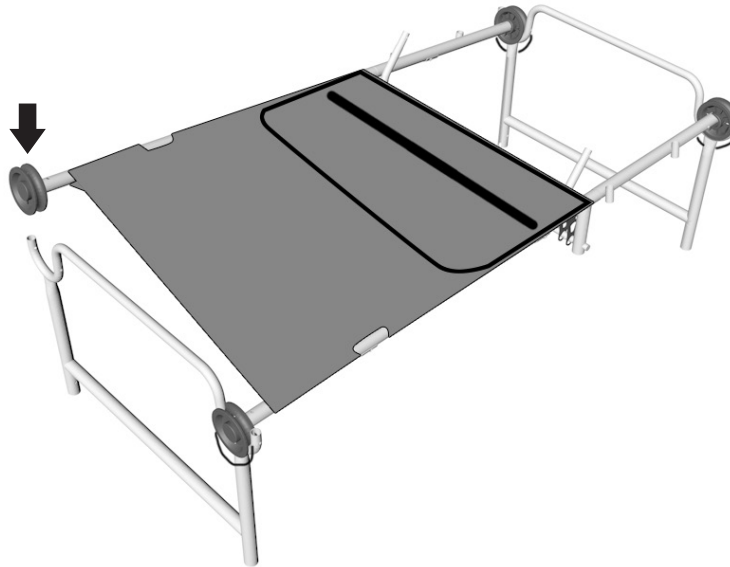
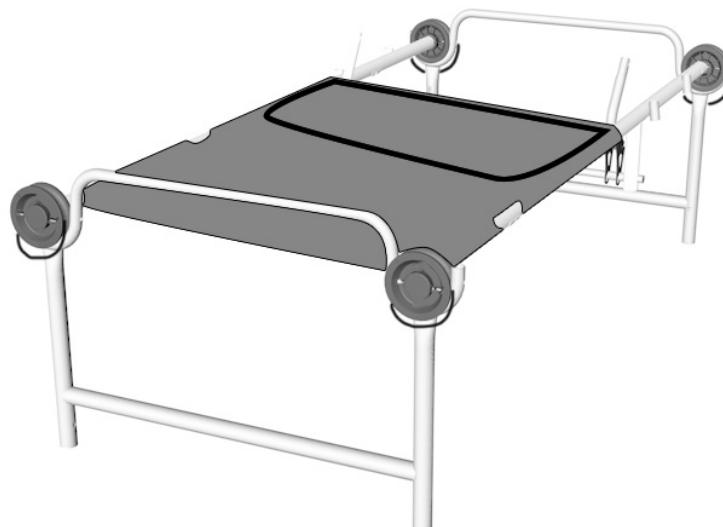
9.1 Insert back rest into backrest sleeve.

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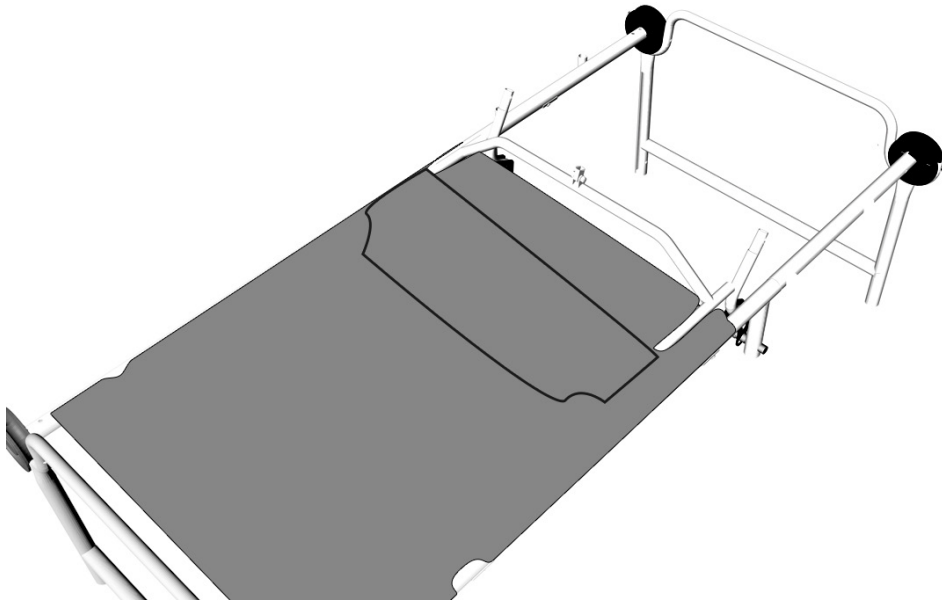9.2 Making sure that the support frames from the rails are through the loop, connect backrest to hinge of side rails. Click into place.

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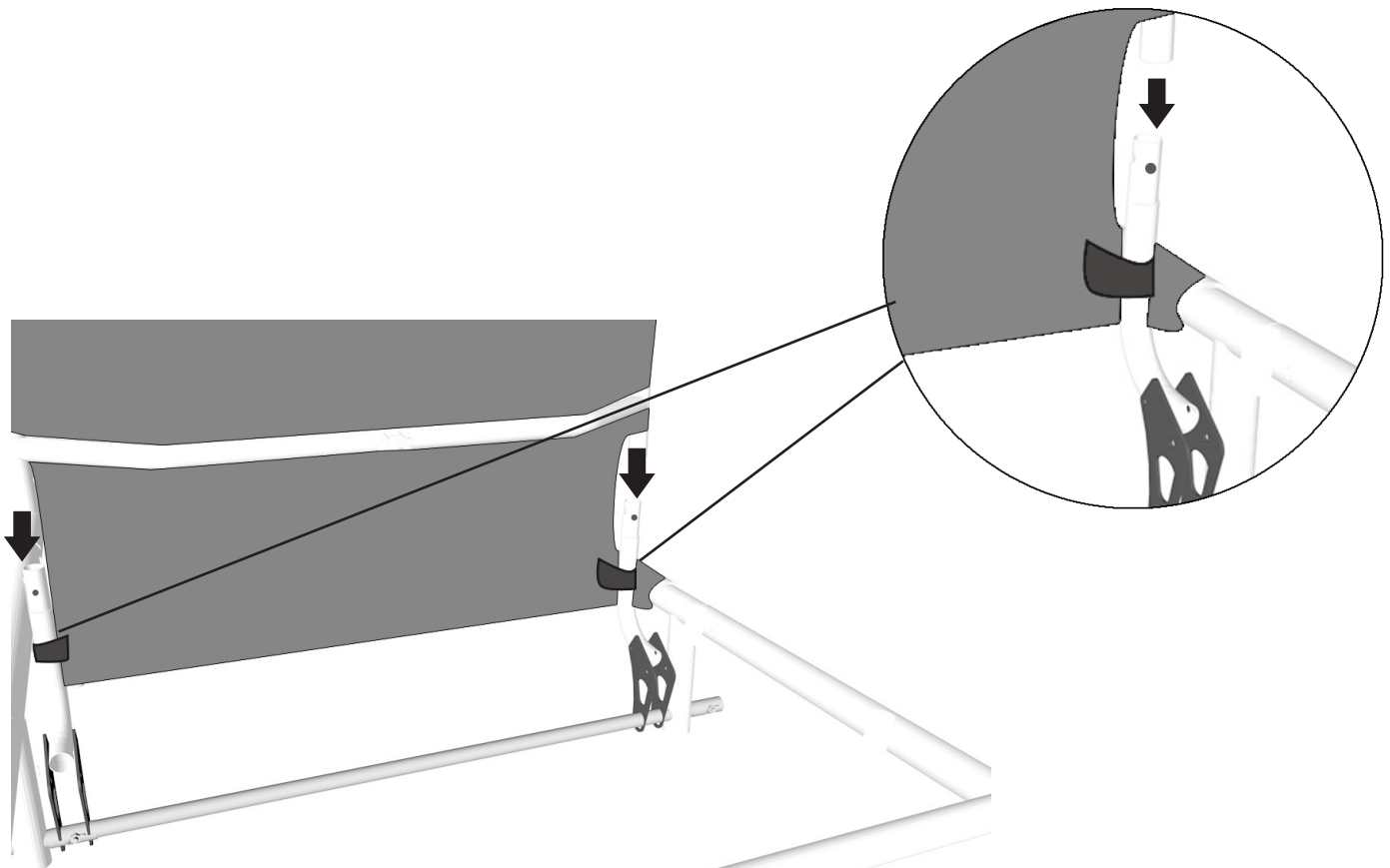
STEP 10. CONNECTING STABILIZER BRACKETS

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- 10.1 Line up stabilizer bar with the attachment points to the leg of the head end frame, and to the side rail on side A.
- 10.2 Loosely attach the bolts by hand for side A.
- 10.3 Repeat step 10.1 and 10.2 on side B. at the head end frame.
- 10.4 Repeat step 10.1 and 10.2 on side A. at the foot end frame.
- 10.5 Repeat step 10.1 and 10.2 on side B. at the foot end frame.
- 10.6 Go back and tighten the bolts on both contact points on all 4 corners.

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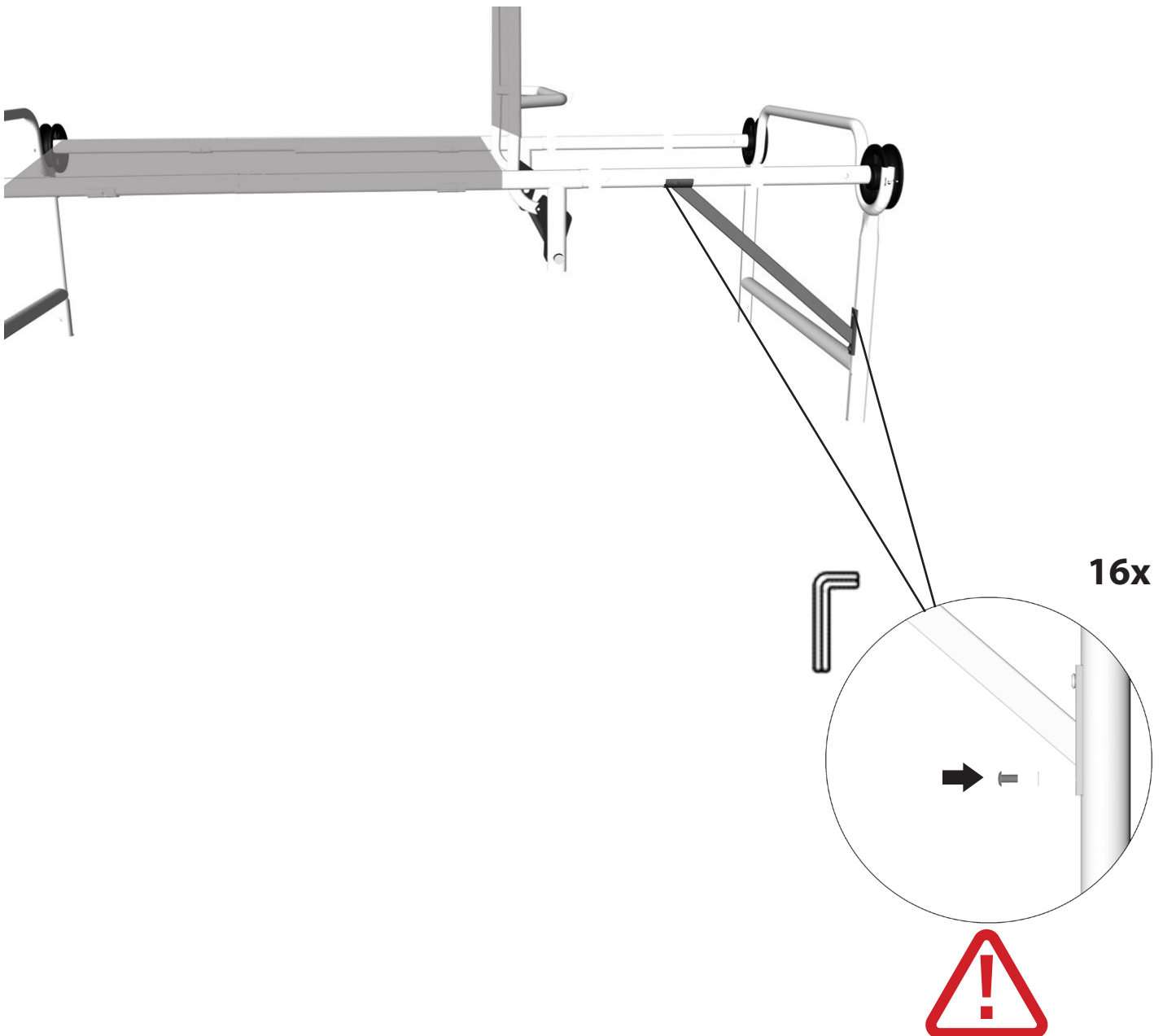
STEP 11. CONNECTING THE PNEUMATIC STRUT SCHRITT 11. GASDRUCKFEDER MONTIEREN


- 11.1 Flip the assembled cot onto its side with side B laying on the ground.
- 11.2 Connect the pneumatic strut to the bottom of the headrest cross bar with the provided bolt.
- 11.3 Use the provided wrench to tighten into place.

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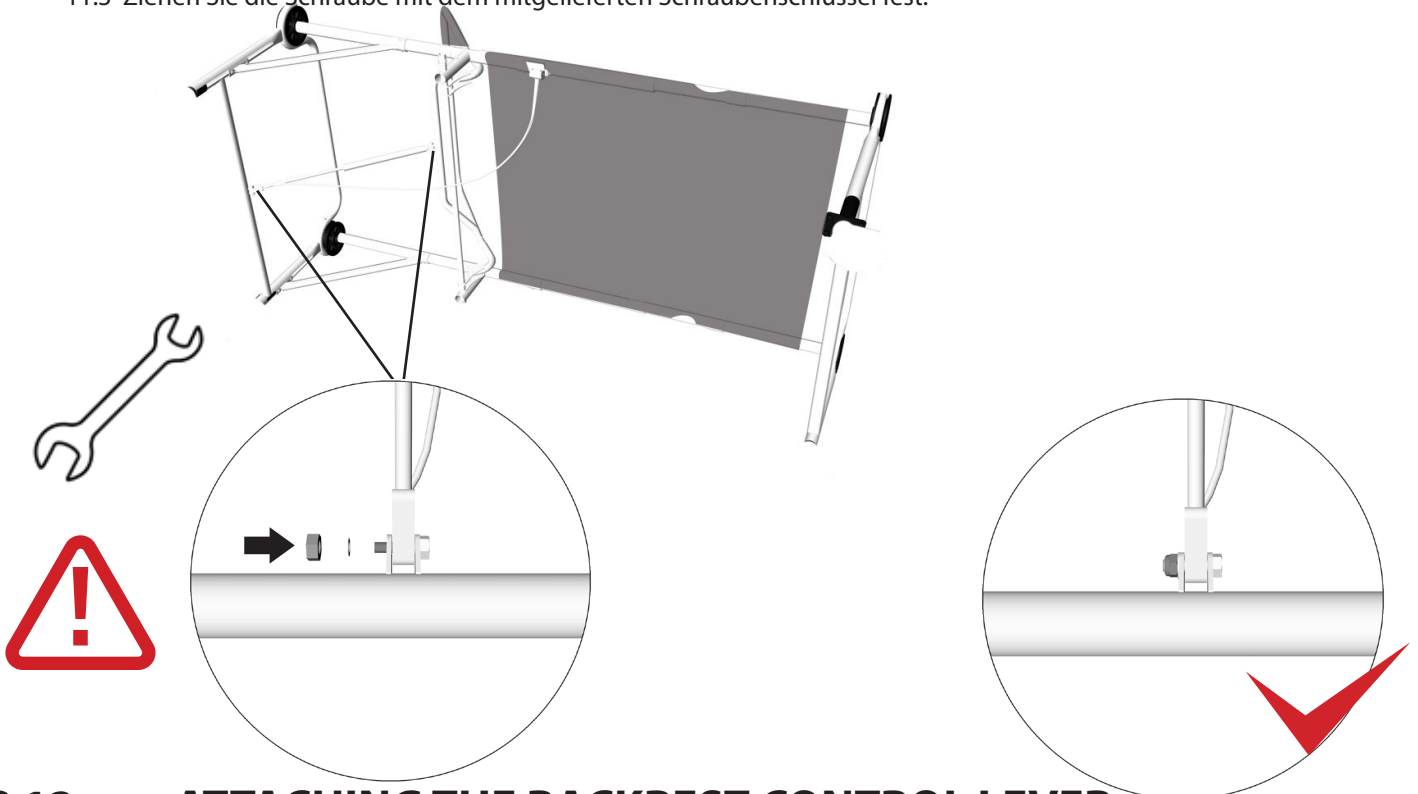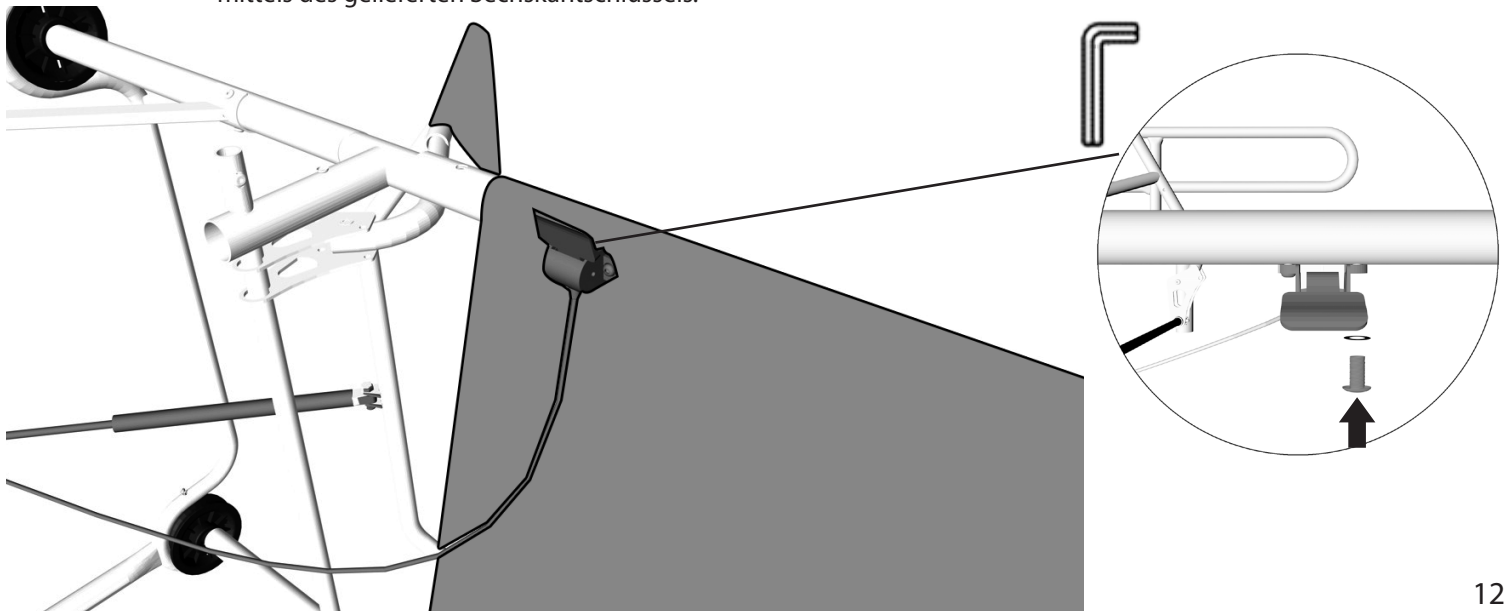
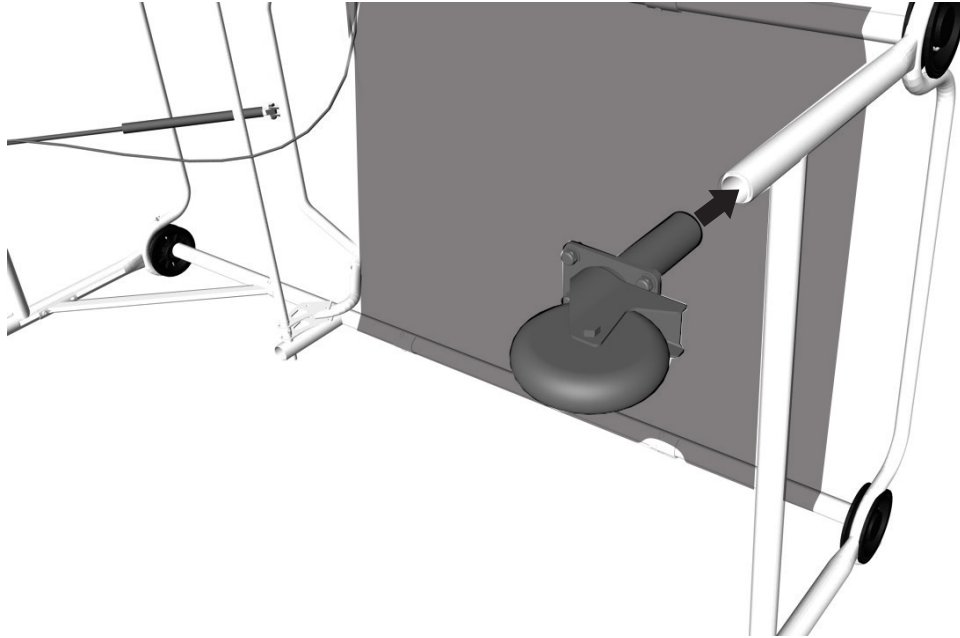
13.1 Insert the wheel into the leg of the bed frame ensuring that the pin locks into place.
13.1 Stecken Sie das Rad in den Fuß des Rahmens.

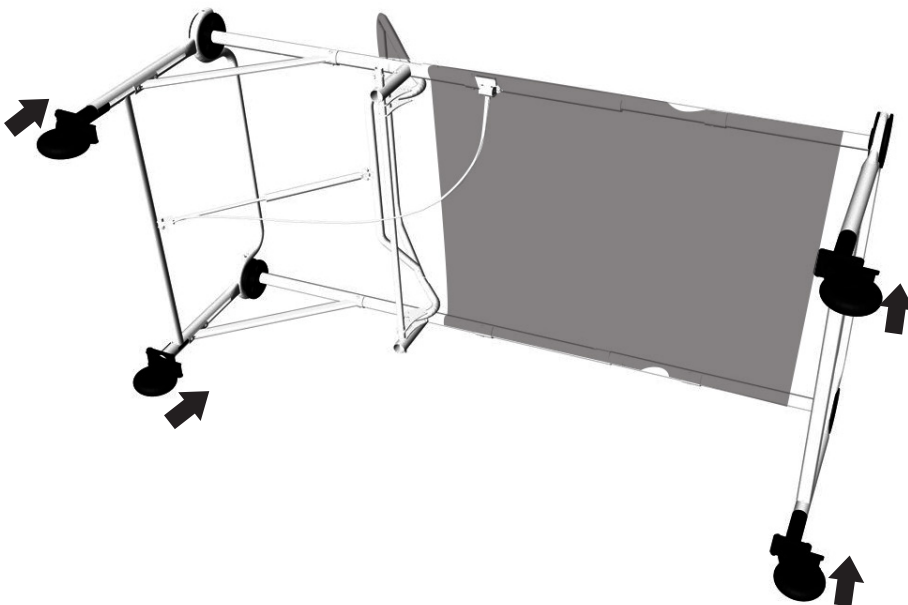13.2 Repeat on all corners.
13.2 Wiederholen Sie dies an allen Ecken.


Make sure each wheel slides into each leg completely and that the pin has locked on all 4 corners. Pin is only securely locked into place when you hear a “click” and the pin has popped up.

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

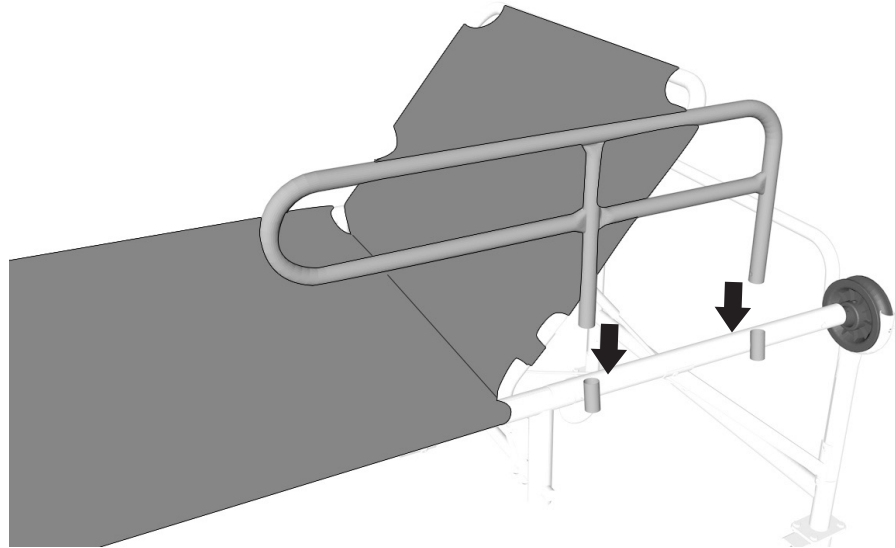
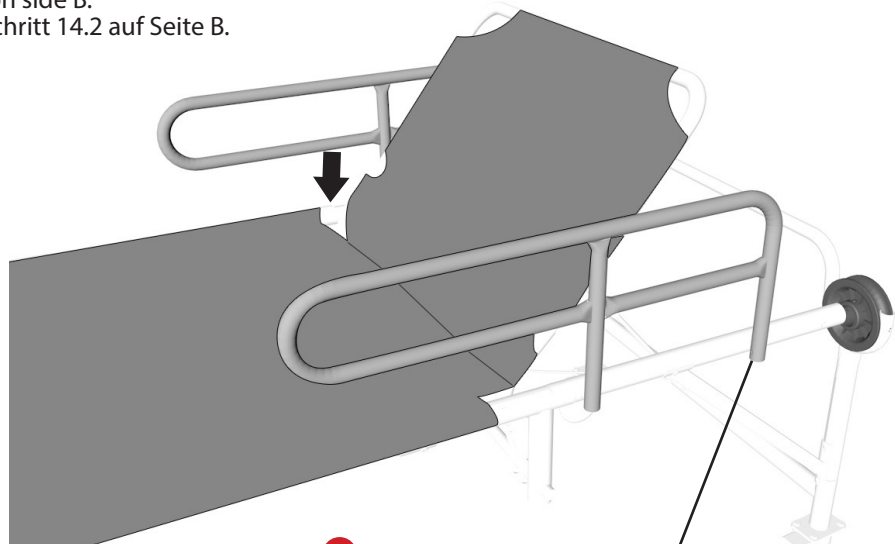
Pin is only securely locked into place when you hear a "click" sound. Make sure pin has popped up before moving on.

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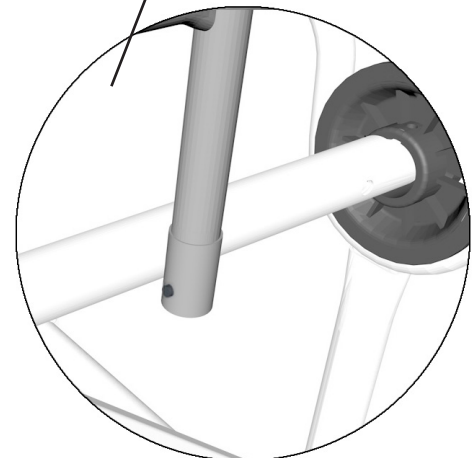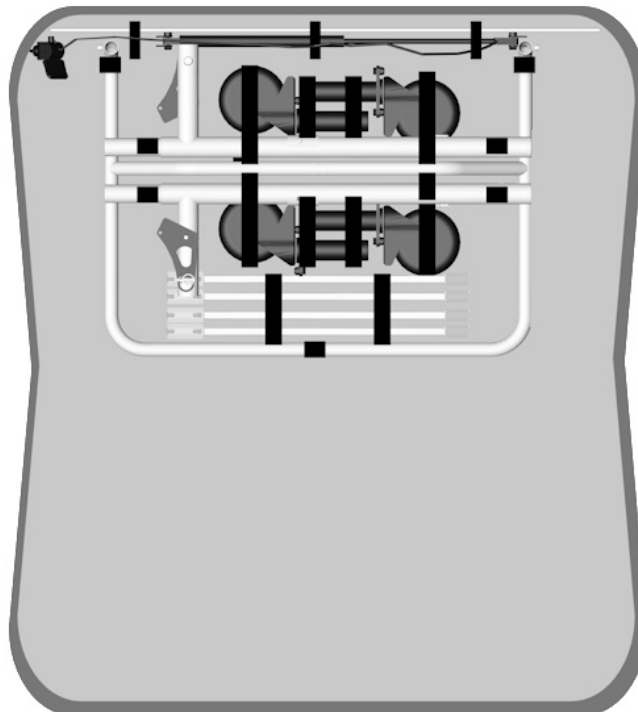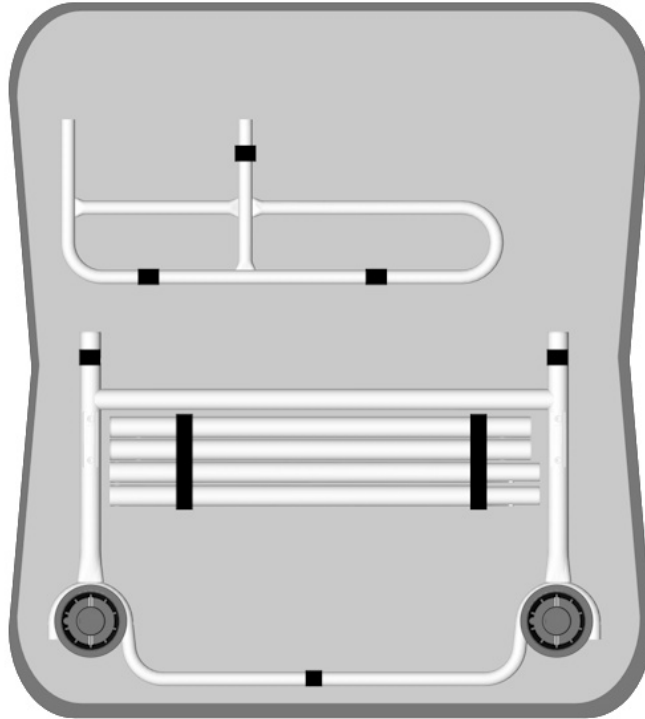Packing Instructions

USAGE & CARE INSTRUCTIONS

1. This product is not designed or manufactured to be used outdoors.
2. Prolonged exposure to the elements will result in rapid deterioration of the product and void any warranty.
3. Ensure product is completely dry and clean before storing in climate controlled dry storage.
4. **Hand wash only**:- Clean the fabric and frame using a soft cloth and a mix of soft detergent and warm water – do not use bleach or any other abrasive cleaners. Simply air dry

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WARRANTY CONDITIONS

Our products are subject to strict quality controls. If a DISC-O-BED® product purchased from us does not work properly, we regret this very much and ask you to contact us at the contact address below at www.discobed.de. In addition to the statutory warranty, we grant you a guarantee on all DISC-O-BED products purchased from us, in accordance with the following provisions:

YOUR LEGAL RIGHTS ARE NOT RESTRICTED BY THIS. (a) The guarantee period is 2 years from the date of purchase. The guarantee applies to the territory of the European Union and Switzerland. (b) During the warranty period, products that are defective due to material and manufacturing defects will be repaired or replaced at our option. Replaced parts become our property. Effecting the guarantee neither extend the guarantee period, nor do they start a new guarantee. (c) Warranty claims must be made immediately after becoming aware of the defect within the warranty period.

(d) To assert the guarantee, please contact the service team by email, fax or telephone. Our contact details are available at www.discobed.de.

If the defect is within the scope of our warranty, you will receive a repaired or new product. The return costs are covered by us. (e) Warranty claims are excluded for damage caused by: improper or improper treatment; Environmental influences (moisture, heat, overvoltage, dust etc.); Failure to observe the safety precautions applicable to the device; Failure to observe the operating instructions; Use of force (e.g. blow, shock, fall); Interventions that were not carried out by the service address authorized by us; unauthorized attempts to repair; Sending in non-transport-safe packaging.

(f) Defects or damage to the product that are not (or no longer) covered by the scope of guarantee will be remedied against reimbursement of costs.

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SATISFACTION GUARANTEED

We hope that you are completely satisfied with our product and it is exactly what you expected. Should you have any questions, comments or concerns, please don't hesitate to contact us via:

+49-7621-1630260

info@discobed.de

www.discobed.de

Like us on FB <http://www.facebook.com/discobedeurope>

All returns of defective items shall be made to the following address:

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Disc-O-Bed EU GmbH